



The Best Total Wound Care & Infection Control

First Aid

Shelf Life ~ 4 years

Road Rash

Patient: 6 y.o. – male Belgian Shepherd

This dog sustained a deep open wound as a result of car accident. The depth and location of the wound made it impossible to suture.

Derma GeL[®] Treatment

Derma GeL[®] was applied - during the first 72 hours - once a day covered by a non-adhesive dressing and with a cohesive bandage. The fourth day, Derma GeL[®] was applied – uncovered - twice a day. After 3 weeks, the wound healed completely and without scarring. The patient is able to move freely without any loss of mobility.

Key Points:

- Fast healing (60% faster than any other product)
- Protective film effect + insect repellent
- GeL stayed where applied (vertically)
- No infection
- No suturing
- Minimal bandaging (72 hours only)
- Excellent patient acceptance
- No loss of mobility (no tissue adherence)
- Best value for money (fast healing = less applications)

